


SIZING CHART

Guidelines Based on Body Measurement

Measurement Instructions

Have the patient remove any outerwear like a sweater or jacket and have them stand straight with arms at their side. The patient should not inhale to overly expand their chest, but be in a natural state. Using a tailor's or sewing tape, take the chest/bust measurement under the arms and across the **largest** part of the chest, and do the same around the largest part of the abdomen. Pull the measuring tape around without stretching or tightening the tape, a relaxed measurement. Take the chest/bust and abdomen measurements with the patient's arms at their side (not with their arms up or outwards, see drawing below). Use the larger of the two measurements to determine correct size. If there is a greater than 12" (30 cm) difference between the chest/bust and abdomen, please contact IBC for sizing determination assistance.

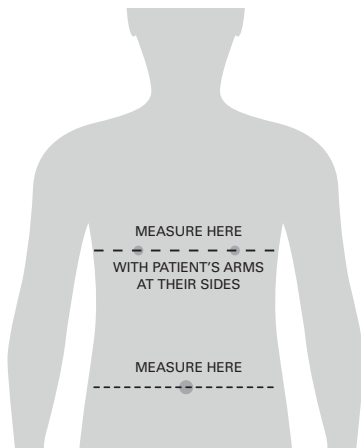

SIZING CHART

Size	Body Measurement
XXS	18" – 23" (46 – 58 cm)
XS	23" – 29" (58 – 74 cm)
S	29" – 35" (74 – 89 cm)
M	35" – 41" (89 – 104 cm)
L	41" – 48" (104 – 122 cm)
XL	48" – 55" (122 – 140 cm)
XXL	55" – 65+" (140 – 165+ cm)

Please note:

- Fit and sizing vary from person to person.
- Using the front adjustment straps, the AffloVest should fit very snugly on the chest, but not limit the patient's ability to take in a full, deep breath.
- If needed, adjust the shoulder snaps to place the front upper AffloVest oscillation motors on the upper chest just below the collar bone.

Upon receipt of AffloVest, please note:

1. A 1 inch (2.5 cm) overlap of the vertical black seams on the front of the AffloVest is acceptable for all sizes.
2. A 1 to 5 inch (2.5 – 13 cm) gap between the two sides of the AffloVest is also acceptable for all sizes.



Patents Pending – USA & Worldwide